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Electronic Baby Scale

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ENGLISH

You have purchased a RHINO product, which guarantees quality and durability, since the BABE-25 scale was manufactured according to American quality standards. Keep this instruction manual, it will help you understand the operation of the RHINO BABE-25 scale will be useful for future reference.

TECHNICAL SPECIFICATIONS

ACCURACY

Medium III

CAPACITY

Máx= 25 kg    Mín= 100 g  
d=e= 5 g    d<sub>T</sub>= 5 g  
Lím= 37.5 kg    T= 25 kg

POWER

6 V---  
(4 batteries "AAA" de 1,5 V---)

TEMPERATURE

32 °F - 104 °F

GET TO KNOW YOUR SCALE

| 1                  | 2      | 3                  |
|--------------------|--------|--------------------|
| HOLD UNIDAD button | Screen | ON/OFF CERO button |

WARNINGS

In case of need of technical assistance, please contact one of the authorized Rhino service centers.

Do not exceed then capacity of the scale and do not heavily drop the product to be weighed, this could damage the loadcell and affect the guarantee of the product.

Make sure that the sticker of guarantee is not broken before returning the product.

CAPACITY

Máx= 5 kg    Mín= 20 g  
d=e= 1 g    d<sub>T</sub>= 1 g  
Lím= 7.5 kg    T= 5 kg

Do not leave then weight on the scale for longer than necessary as this may effect the precision.

Do not open the scale yourself, the warranty will be void.

POWER

(1 lithium battery, type CR2032 included)

Do not use the scale in places with excess of humidity, this could effect the scale.

The scale should not be exposed to dust or splashing from liquids.

TEMPERATURE

32 °F to 104 °F

## CAUTIONS

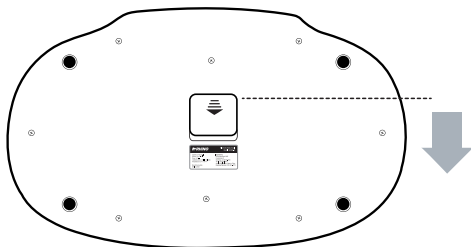
In case of need of technical assistance, please contact one of the authorized Rhino service centers.

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
## BATTERY INSTALATION

- Take the battery cover and carefully pull it up.


- Place the new batteries on the scale.
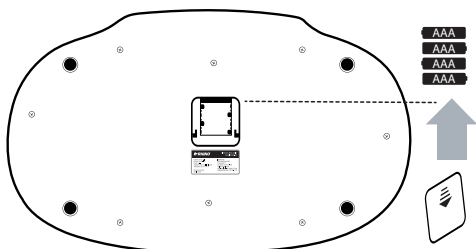
## WEIGHING PROCESS

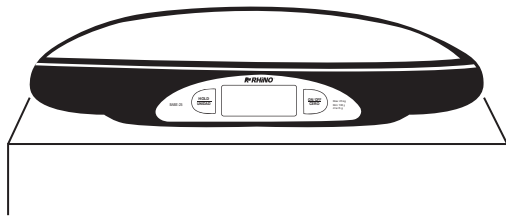- To ensure weighing accuracy, please place the scale on a solid, flat and firm surface.

- Press the **ON/OFF ZERO** button to turn on the scale, if required place a blanket and press the **ON/OFF ZERO** button again before placing the baby to discard the weight of the blanket and slowly place the baby on the scale.

**NOTE.** If when removing the baby from the scale, the screen registers weight, press the **ON/OFF ZERO** button to restart the scale.

- The screen will show the baby's weight, press the **HOLD/ONWEIGHT** button to keep the baby's weight fixed on the screen. The weight will be showed 2 times as confirmation and will be stored in the scale's memory
- When the scale is used again and if you want to compare the baby's current weight with the last stored weight record, press **HOLD/ONWEIGHT**, the screen will show the current weight and the weight difference with an arrow. If the arrow points up, it indicates an increase in weight; the down arrow indicates that the weight decreased with respect to the last record.

**NOTE.** The previously stored weight will be erased when the batteries were changed.

## TURNING ON / OFF

For long battery life, the baby scale will turn off automatically after 2 minutes of inactivity or you can press the **ON/OFF ZERO** button for 3 seconds to turn off the scale.

## UNIT CONVERSION

The scale allows you to change the unit of measure from KILOGRAMS to POUNDS and vice versa.

- To change the unit of measurement press the **HOLD/ONWEIGHT** button, approximately 3 seconds.

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## WARRANTY

Rhinob Maquinaria warrants to customers who purchase any Rhinob products from an authorized seller that Rhinob products will be free of defects in workmanship for the duration of 2 years from the date of purchase.

During the 2 years warranty period, Rhinob Maquinaria will replace the product that has a manufacturing defect as long as it has been determined by Rhinob Maquinaria.

This warranty does not apply to defects caused by misuse or alterations of the product, nor does it apply if the damage was caused by repairs or service provided or attempted by anyone other than Rhinob Maquinaria.

To make this warranty effective, you must first contact our Customer Service by dialing (619) 393-1079 or by mail to info@rhinoglobal.net, once you have contacted our Customer Service they will request from you a warranty claim by email.

Rhinob Maquinaria will review the warranty claim. If the warranty claim is found to be valid, please return the product with a copy of the invoice and order ID to the of the invoice and order ID to the indicate address.

Upon receiving the product, Rhinob Maquinaria will sent a replacement product to customer's address.

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**Rhinob Maquinaria S.A. de C.V.**